I. Introduction

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I. Introduction

Thank you for choosing the GD Cleaner by C2S. This machine with its patented system will enable you to **clean removable dental appliances** and keep them clean with the utmost simplicity and safety.

The GD Cleaner by C2S has been designed and **developed by oral health specialists** to improve the comfort and oral hygiene of denture wearers.

This innovative, quick, low-cost system **which cleans dentures by gentle brushing using pins of a special alloy (stainless steel 304) has been perfected for the regular cleaning** of all kinds of removable or threaded dental prostheses or implant pillars, and helps dental prostheses stay clean and last longer.

The use of **autoclavable tubs** and **individual blister packs of cleaning pins** optimizes oral hygiene, avoids cross-contamination and promotes wearer comfort.

WARNING: THE GD CLEANER SYSTEM AND ITS ACCESSORIES MUST BE KEPT AWAY FROM CHILDREN AND FROM ANYONE WITH A PACEMAKER.

Pack Contents: GD Cleaner by C2S (D-C2S-001)

1. One machine (GD Cleaner by C2S)
2. Two magnets for retention of the pins in the tub (8)
3. One power adaptor
4. One User Guide (leaflet or online).

C2S-Clean tartar removal solution pack (D-C2S-004)

Pack Contents: 2 autoclavable cleaning tubs (D-C2S-002)

1. Two autoclavable tubs for the GD Cleaner
2. One User Guide

Pack Contents: 20 stainless steel pin kits (D-C2S-003)

1. 20 blister packs (40g each) of non-reusable cleaning pins
2. Patient reference labels

II. Product description

III. Instructions for use


1/ Powering on

- Connect the power supply lead to the adaptor.
- Connect the power supply lead to the machine and to a standard power socket.
- Switch the GD Cleaner on using the O/I switch at the back of the machine (2).
- The machine automatically goes into standby if not used for 15 minutes. press either button once to bring it out of standby.
- When you have finished, switch the GD Cleaner off completely using the O/I switch located at the back of the machine (2).

2/ To prepare the cleaning tub

- Open the blister pack containing the metal cleaning pins directly into the tub ①. For greater security, position the pin retention magnet under the tub ②.
- Close the tub ③. (Fig.1)

- Pin tracking for patients: Each blister pack of pins bears a detachable tracking label. This can be stuck into the patient's individual file to keep a record of treatment.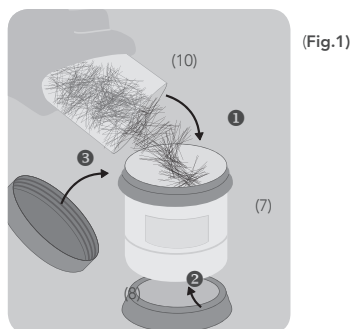

If by mischance the pins should fall out of the tub (because you have forgotten to use the pin retention magnet, for instance), they must not be picked up by hand nor by using the retention magnet. The manufacturer declines all liability for any problems arising from the hazards of doing so (bacteria on the pins, injury, &c.).

4/ Programming and starting the cleaning process

- Press the "start/pause" button.
- You can stop the cleaning process at any time by pressing the "start/pause" button.
- During the cleaning process, the motor will stop for 10 seconds every 3 minutes.
- Do not remove or open the tub during the cleaning cycle.
- Set the time by using the "up/down" arrows. The timer is pre-set to 2 minutes. The time decreases or increases by 2, 5, 10, 15, or 20 minutes.
- The cleaning time selected depends on the build-up of material on the dentures or implant pillars. About 2 minutes for implant pillars. Between 5 and 10 minutes to keep dentures clean (normal build-up). Between 10 and 15 minutes for dentures with medium build-up of dirt. Between 15 and 20 minutes for dentures with heavy build-up of dirt.

5/ Recovering the prosthesis at the end of the cleaning process

- Remove the tub from the machine.
- Position the retention magnet ①.
- Open the tub ②.
- Take the prosthesis out, wearing gloves ③.
- Rinse the prosthesis thoroughly with copious amounts of water.
- Dry the prosthesis with absorbent paper, or put it in a glass of water with a mouthwash solution to rinse it before returning it to the mouth.

Note: after a long period of cleaning the water in the tub could be slightly hot. This is quite normal: the heating is caused by friction among the pins during the cleaning process.


NB: CHECK CAREFULLY THAT THERE ARE NO METAL PINS ON THE PROSTHESIS!

NB: THE TUBS ARE ONLY TO BE USED FOR THE GD CLEANER. THE TUB SHOULD BE USED FOR NO MORE THAN 12 MONTHS.


3/ Use of the cleaning tub with GD Cleaner

- Before the tub is opened it should be placed on the pin retention magnet so as to keep the pins in the bottom of the tub ①.
- Open the tub.
- Fill the tub with 50ml of C2S-Clean (provided by our distributor Global D), then top up with water to the level of the marks on the inside and outside of the tub ②.
- the use of this product is recommended by C2S, as it significantly increases cleaning effectiveness. (Fig.2)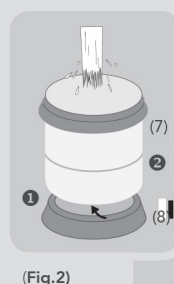

NB: USE ONLY THE FLUID SUPPLIED BY GLOBAL D. THE C2S-CLEAN SOLUTION DOES NOT HAVE A SAFETY CAP, AND MUST BE STORED OUT OF THE REACH OF CHILDREN.

- Wearing gloves, put the dental prosthesis into the tub, teeth down ③.
- Close the tub ④, remove the pin retention magnet ⑤ and insert the tub in the GD Cleaner. (Fig.3)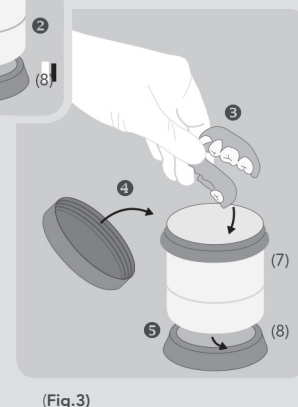

6/ Rinsing and sterilizing the tub each time after use


- The pins must not be re-used
- Keep the pin retention magnet under the tub.
- The liquid contained in the tub should be poured off down a sink or into a waste water container. ①
- Refill the tub with water and stir the pins using the magnet with a small circular motion. This will allow removal of post-cleaning residues.
- Remove the magnet so that the pins can be turned out into a contaminated waste container. (Fig.5) ②
- Tip: this is easier if you use a sieve to catch the pins before disposing of them. (Fig.6)
- Sterilize the tub in an autoclave each time after use.

NB: REMOVE THE SOUND-DAMPING DISC. (9)

- Change the cleaning tub once a year.

NB: use a damp cloth to clean the GD Cleaner. Do not use solvents or corrosive products.

- used tubs and metal pins must be disposed of in the appropriate containers: the pins in the Sharps container and the tub in the Soiled Items container.
- Do not reuse the pins, either in another tub or for anything else.

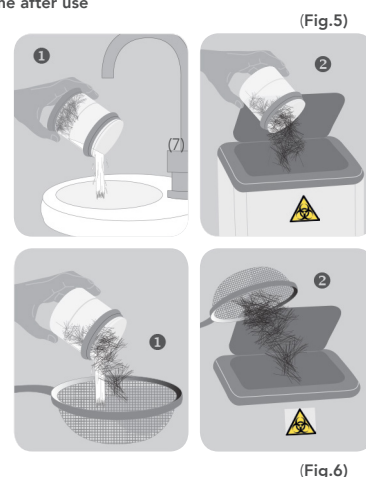(Fig.6)

See www.globald.com for further details.

WARNINGS

- This appliance may be used by adults, including those lacking experience and expertise or with diminished physical, sensory or mental capacities, provided they are properly supervised or instructed in safe use of the appliance and provided they have understood the risks involved.
- The GD Cleaner should not be used to clean appliances which are damaged (cracked, broken, &c.).
- The GD Cleaner system should be stored in a dry place.
- Avoid prolonged exposure of the GD Cleaner system to sunshine or other heat sources.
- Clean the GD Cleaner with a damp cloth. Do not use solvents or corrosive products.
- Cleaning and user maintenance must not be carried out by children without supervision.
- Power the GD Cleaner off completely when you have finished with it, using the "O/I" switch at the back of the machine.
- The appliance should only be used with the power adaptor supplied with the appliance.
- If the power lead is damaged it must be replaced, either by the manufacturer, the manufacturer's after-sales service agent or a similarly qualified person so as to avoid any hazard.
- The external flex of this adaptor cannot be replaced; if the lead is damaged the adaptor should be disposed of and replaced by one of the same model.
- The cleaning tub may only be used with the GD Cleaner system.
- The GD Cleaner system is designed to be used only with the tubs and cleaning fluid supplied by GD. Global D and C2S decline all liability for any problems arising from use of products other than those supplied by Global D.
- The metal cleaning pins must never be re-used.
- Neither tub nor pins should be used (together or separately) otherwise than for their purpose as part of the GD Cleaner system.
- The tub and cleaning pins, and the whole GD Cleaner system, are intended only for the cleaning and maintenance of fixed or removable dental prostheses and implant pillars.
- The cleaning tub must be changed once a year.
- Do not remove or open the tub during the cleaning cycle.
- If pins should fall out of the tub (because you have forgotten to use the pin retention magnet, for instance), they must not be picked up by hand nor by using the retention magnet. The manufacturer declines all liability for any problems arising from the hazards of doing so (bacteria on the pins, injury, &c.).

Due to our policy of constant product improvement the machine may not exactly resemble those in the photos

Distributed by Global D
Zone Industrielle Sacuny-BP82
118 avenue Marcel Mérieux
69530 BRIGNAIS - FRANCE
www.globald.com

Made by C2S
74 rue Maurice Flandin
69003 LYON - FRANCE
contact@c2s-medical.com