

Medication preparation range

Severo® electrical pill crusher

In order to facilitate medication intake, medical staff must regularly crush the pills for the patients who have swallowing disorders.

In order **to avoid musculo skeletal disorders** due to manual crushing, the electrical Severo® crusher has been developed in cooperation with health professionals.

This crusher which has already been approved by many establishments, enables **to reduce hard medicines into fine powder** in less than 8 seconds.

Photos are not contractual



Hygienic



Ergonomic

Advantages

- ▶ Simple, efficient and robust
- ▶ Low noise
- ▶ Crushing without any effort neither MSD
- ▶ Can easily be cleaned
- ▶ Light and handy



Video

Severo® electrical pill crusher

Technical features

- ▶ Dimensions: L. 80 x D. 70 x H. 217 mm
- ▶ Autonomy: 500 crushing cycles (≈ 3 day permanent use)
- ▶ Supplied by rechargeable battery (adaptator is included)
- ▶ Weight: ≈ 1,5 kg
- ▶ Noise level: 52 db
- ▶ Loading time: 2 hours
- ▶ Warranty: 2 years for device and 1 year for battery



Description

Reference

Severo® electrical pill crusher

NA00030.01

Severo® electrical pill crusher + stabilizer

NA00030_STAB

Severo® electrical pill crusher + stabilizer + cup holder

NA00030_DIST

Use



Open the Severo® lid by pressing on the metallic button.



Place the cup on the tray.



Put the medicines into the cup until the first line maximum.



Place a washer on the cup.




Place the crushing head on the washer.



Close the Severo® lid by light pressure.



Press the power supply key so that the crushing process starts. The LED light of the key turns to green once crushing is finished.

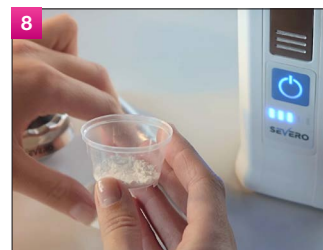

Open the Severo® lid. Lift up the crushing head and the washer. The medicines are reduced to fine powder and can be mixed with patient's meal or the Swallowing gel.

Medication preparation range

Severo® electrical pill crusher

Severo® stabilizer

- Dimensions: L. 160 x D. 160 x H. 4 mm
- Weight: 150 g
- Steel base with felt-covered underside
- Adhesive magnet to attach to the bottom of the Severo®

Description

Reference

Severo® stabilizer

NA00040



Cup holder

- Capacity: 35 cups
- Material: White epoxy-polyester coated steel wire
- Mounting method: Magnetic attachment to the stabilizing plate
- Dimensions: L. 54 x D. 46 x H. 155,5 mm

Description

Reference

Cup holder

NA00021



Consumables

Description

Reference

2100 single use cups

NA00014.01

2000 single use lids for cups

GV60281

2100 washers single-use

NA00016



Severo® gelled water

Severo® gelled water is designed to facilitate medication administration. It is ideal for people with swallowing difficulties.

Description

Packaging

Reference

4 bottles of tutti frutti flavored gelled water with dosing pump

4 x 1 liter

NA00047

1 sample of tutti-frutti-flavored gelled water

8 mL

NA00043

4 bottles of banana-flavored gelled water with dosing pump

4 x 1 liter

NA00048

1 sample of banana-flavored gelled water

8 mL

NA00046

